



MAIL THE
RETURN BOX

INSTRUCTIONS

STD-TEST FOR MEN
Chlamydia and Gonorrhoea

You can take the test as late as 14 days after you had (unsafe) sex for the last time.

STD-TEST

You can use this sampling kit to test for Chlamydia and Gonorrhoea. Your material will be examined at our laboratory (ISO15189). The results are authorised by a medical microbiologist. You can ask for the result by using the unique code on the code card.

RESULT

You can ask for the result at <https://testalize.me/uitslag> within 48 hours. To that end, use the code on the code card.

SCAN ME



KEEP YOUR CODE CARD IN A SAFE PLACE. NO CODE? NO RESULT!

CONTENTS OF THE BOX




return box



instructions



code card

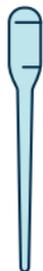

pipette*



vial**



sealbag***

* Pipette | Hologic Inc.

** Aptima® Urine Specimen Transport Tube | Hologic Inc.

*** Sealbag Daklapack

HOW TO CONDUCT THE TEST

Leave the fluid in the vial and avoid skin contact. Do not ingest.
Testing is simple. There are only four steps to complete:

- 1 Urinate into a cup. Make sure you did not go to the toilet for at least an hour. Take a clean cup (such as a disposable cup) and catch the first drops/flow (no more than 30ml).
- 2 Open the vial. Use the pipette to transfer 2ml of urine to the vial. The urine should reach up to between the black lines.
- 3 Securely close the vial. Place the vial in the sealbag and seal the bag. Place the full sealbag in the return box and close the box.
- 4 You can now mail the return box in a PostNL **post box** (testalize.me/retourbox). After the return box is delivered, you can ask for your result at testalize.me/uitslag within 48 hours, using the code on your code card.

* do not collect more than 30ml of urine and transfer the urine to the vial within an hour