



You can ask for the result at <https://testalize.me/uitslag> within 48 hours. To that end, use the code on the code card.

RESULT



You can use this sampling kit to test for Chlamydia and Gonorrhoea. Your material will be examined at our laboratory (ISO15189). The results are authorised by a medical microbiologist. You can ask for the result by using the unique code on the code card.

STD-TEST

INSTRUCTIONS

MAIL THE RETURN BOX




STD-TEST FOR WOMEN

Chlamydia and Gonorrhoea

You can take the test as late as 14 days after you had (unsafe) sex for the last time.

CONTENTS OF THE BOX

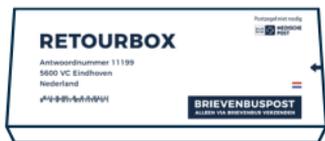

return box



instructions



code card



cotton swab*



vial**



sealbag***

* cotton swab: Aptima® Multitest Swab (SW) CE0843 MDD 93/42/EEC | Hologic Inc.

** vial: Aptima® Multitest Swab Transport Medium (STM) | Hologic Inc.

*** sealbag Daklapack

HOW TO CONDUCT THE TEST

Leave the fluid in the vial and avoid skin contact. Do not ingest.
Testing is simple. There are only four steps to complete:

- 1 Wash your hands. Remove the cotton swab from its packaging. Do not touch the cotton and make sure it is not exposed to dirt.
- 2 Separate your labia with one hand*. Hold the cotton swab on the snap line. Insert the cotton swab into your vagina up to 5 cm. Turn it around for 10 to 30 seconds and move it along the walls of your vagina.
- 3 Open the vial. Place the cotton swab inside and snap it along the line. Securely close the vial. Place the vial in the sealbag and seal the bag. Place the full sealbag in the return box and close the box.
- 4 You can now mail the return box in a PostNL **post box** (testalize.me/retourbox). After the return box is delivered, you can ask for your result at testalize.me/uitslag within 48 hours, using the code on your code card

* take a comfortable position as you would when you insert a tampon